EN

Important: US federal law limits the sale of this product by or under the order of a physician.

Description

The RocaStone and Kobol stone extractors are sterile and for single use. Refer to the packaging for the specifications of each extractor. All the baskets of the products in this range are made of Nitinol.

In the distal portion, the basket and the guide pass through a flexible shield enabling progression into the interventional channel and the different functionalities of the basket. In the proximal portion, the stone extractor has an ergonomic handle at the end enabling control of the movements of the basket.

For rigid ureteroscopy you can use either tip or tipless stone extractor basket.

For flexible ureteroscopy and/or ureterorenoscopy the recommendation is to use a tipless stone extractor basket.

Indications

Kobol and RocaStone Extractor stone extractors are intended for grasping and/or manoeuvring and/or extraction of stones during ureteroscopy and ureterorenoscopy procedures.

Contraindications

Objects may be too large to be removed endoscopically with an extractor. To avoid stone impaction, the physician should examine the patient to determine the size of the object. Do not use the extractor if the object is too large to be removed endoscopically or to be all held within the extractor.

Potential Complications

perforation, entrapment, hemorrhage, incapacity to release an unextractable stone from the basket and basket inversion.

Warnings

- Avoid contact with any electrical appliances, this could damage the extractor.
- Avoid direct contact with a lithotripter, this could damage the extractor.
- Use of the extractor without an introducer may damage it.
- In all cases, avoid any situation that could lead to damage to the extractor. A damaged extractor may cause seious injuries to the patient.

Precautions

- Do not use extractor if damage to the sterile barrier is suspected (i.e. damaged and/or deteriorated packaging).
- Do not use extractor if it has reached or passed its use by date.
- Physician should be thoroughly trained in endoscopic techniques before using this device.
- Extractor should only be used after careful visual inspection for damage by the intervening physician upon opening of the sterile packaging. Do not use extractor if any damage is suspected.
- Prior to the procedure, manoeuvre the device to ensure proper function.
- It is strictly prohibited to try to dismantle, repair or modify this device before use.
- Extractor must be handled carefully to avoid kinking the device.
- Avoid extreme force when opening, closing and removing the stone basket, this will increase the likelihood of kinking and/or damaging the device.
- Use extreme caution when using the device and a laser fiber; a laser can easily damage the device.

Instructions for use

Carefully read all instructions prior to using these extractors. Open sterile packaging and remove the extractor using usual aseptic techniques. On the sterile table, use the handgrip to remove the mainwire and basket from the protective sheath. Inspect the extractor for defects on the sterile table. Ensure that the mainwire and basket are correctly connected to the handgrip before starting the procedure. Do this by manoeuvring the extractor outside of the patient's body in order to guarantee that the extractor is functioning properly (basket expansion, rotation and closure) and that the different parts are correctly interconnected.

NOTE: Do not use the device if any defects or dysfunction are observed during these manoeuvres. If defects are observed, replace the device. Care should be taken not to link the mainwire during these test manoeuvres. If the extractor becomes kinked, it should be replaced immediately.

Pass the distal tip of the extractor into the interventional endoscopic route. The basket should be closed. An introducer may be used for the smaller extractor sizes to facilitate insertion of the distal tip. Then, carefully advance the extractor along the interventional catheter by exerting intermittent gentle pressure while monitoring the appearance of the distal tip of the extractor at the end of the endoscope. Before activating the basket, ensure that it fully goes beyond the interventional channel.

To capture the stone, move the extractor with its basket closed alongside or behind the stone. Open the basket using the handgrip and capture the stone. Rotational movements may help to capture the stone. Once the stone is positioned inside the basket, close the basket using the handgrip. Various techniques can be used depending on the type of stone, its shape and location. When the stone is captured, use the handgrip to keep the basket closed throughout removal of the extractor and stone.

Models with a channel for laser fibre:

Some models comprise a channel allowing to guide a laser fibre in the patient. The models concerned are exclusively those which include a specific mention on the label. They must be used exclusively with a laser fibre of the brand Rocamed. Before use, ensure that the diameter of the fibre does not exceed the maximum diameter indicated on the label of the extractor.

Do not activate the laser while the extremity of the fibre does not exceed the basket, outside of the sheath.

Insert the laser fibre carefully into the attachment mounted on the handle until you see the end of it in the basket. In case of resistance, do not force it, and look for the cause of the blockage. The Tuohy Bost can be tightened to attach the fibre to the instrument.

Conditions of use and storage

The device should be stored and transported in a normal environment, i.e. away from extreme temperatures and humidity. Do not use if the sterile barrier is damaged and/or deteriorated. These devices are for single use only. Do **not** reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the integrity of the device and/or lead to failure, which may result in patient injury, illness or death. Also reuse, reprocessing and resterilization may introduce a risk of contamination of the device and/or cause patient infection or cross infection to another patient, which may lead to injury, illness or death. Under any circumstances, the manufacturer will not be held liable in the case of reuse, reprocessing or resterilizing of the device. After use, dispose of device and packaging in accordance with hospital, administrative and/or local government policy.

Warranty and Liability Statement

Rocamed Inc., Promepia and its Affiliates warrants to the first purchaser of this device that reasonable care has been exercised in the design and fabrication of this device. If any suspected damage is found please call the Rocamed Inc. representative. This limited product warranty is in place of all other warranties, whether express or implies, including but not limited to any implied warranty of merchantability or fitness for a particular purpose. Rocamed Inc., Promepia and its Affiliates under this limited device warranty will be limited to replacement of defective device. **Under no circumstance will Rocamed Inc., Promepia and its Affiliates be liable for any indirect, incidental or consequential damages resulting from your handling or use of this device.**

	 Manufacturer : ROCAMED France – ZI de Signes Allée de Stockholm - 83870 Signes – FRANCE Tel. +33 494 902 100 - Fax. +33 494 986 055	Distributed by ROCAMED SAM 9 Avenue Albert II, 98000 Monaco Tel: +377 97 98 42 43 Fax: +377 92 05 61 50 Email: info@rocamed.com
--	--	--

		This device is not made with natural rubber latex	CE marked on: 21 July 2015
<p>Sterilized using ethylene oxide</p>	Do not reuse		

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